

# FistulaStop replaces manual-pressure after dialysis treatment

## FistulaStop is comfortable for the patient

- Gentle towards fistulas and grafts
- Applies an even pressure every time
- An easy way to stop the blood flow
- Replaces need for patient to press on fistula following dialysis

## FistulaStop provides benefits for the health care workers

- Ergonomic as it replaces pressing by hand
- Easy to clean with alcohol or hot water
- Re-usable
- Can easily be assigned to a particular patient through name-tagged storage box
- One size fits all
- Time saving

## FistulaStop is easy to use

- Place a gauze swab between the access of the fistula and the FistulaStop
- Be sure that the bulge covers the fistula access
- Stretch the strap around the arm and secure it
- Remove FistulaStop when the blood flow has stopped and before the patient goes home
- FistulaStop should be changed once a month (if continuously used)

**FistulaStop is made in Sweden**

**[www.fistulastop.com](http://www.fistulastop.com)**



*fistula***STOP**<sup>™</sup>

### Order information

Product	Order code
FistulaStop 1 pair	F 500

CE

FistulaStop is a patient bound product. It is delivered in pairs in a plastic box with a name tag attached on it.

Design protected reg.no. CDR 001614173